


Fetal Monitor *iCTG*

Quick operation manual

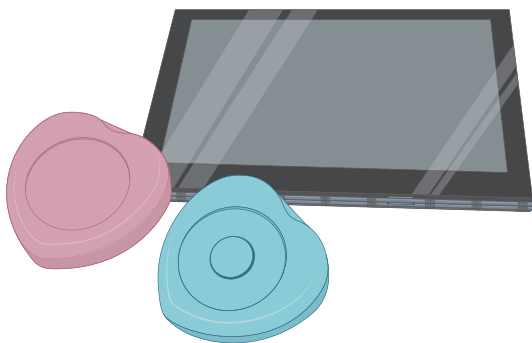

Name of each part and function	2
Start Measurement	4
Finish Measurement	6
Display and send past data	7
Patient selection / registration	8

inquiry



Usage Video



Manufacturer
Melody International Ltd.

〒761-0301
2 2 17-44 Hayashi, Takamatsu, Kagawa
Next Kagawa 304

Tel : 087-813-7362

Fax: 087-813-7361

Support desk

Email: support@melody.international

Caution


Quick operation manual describes only basic usage.
Please use the quick manual after fully understand the content
of the separate operation manual
Please keep operation manual in a safe place.




Melody International

Name of each part and function

FHR Transducer

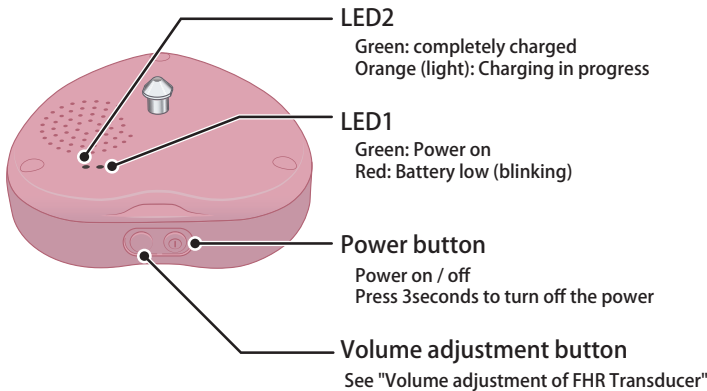


UC Transducer

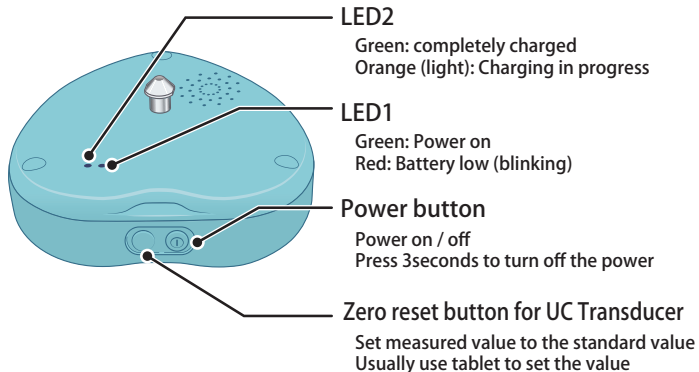

Volume adjustment of FHR Transducer


- **Volume adjustment by the volume adjustment button of FHR Transducer**
 - Each time you press, the volume changes
Large → Medium → Small (→ Large: Repetition)
Press and hold more than 3 seconds or more to mute

- **Volume adjustment by tablet**

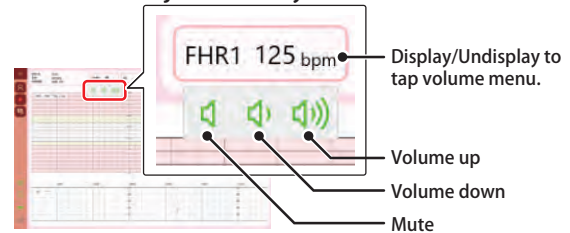


Charging FHR Transducer and UC Transducer

Important It can not be measured while charging

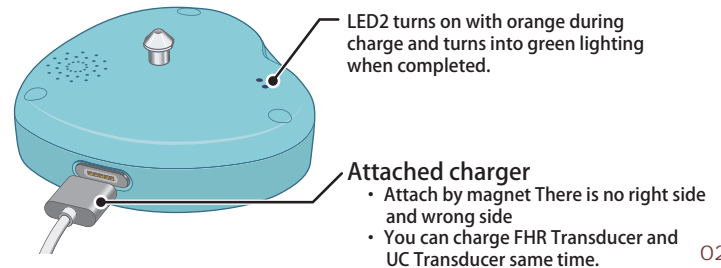

Main screen

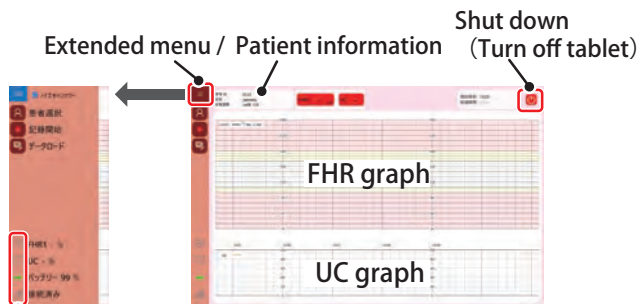


- Display patient list See page 6
- Start measurement
- Display data list See page 7

See "Indicator (icon) display and function status"

- Tap or to display FHR Transducer and UC Transducer
- Tap to display setting screen

Indicator (icon) display and function status

● Battery level

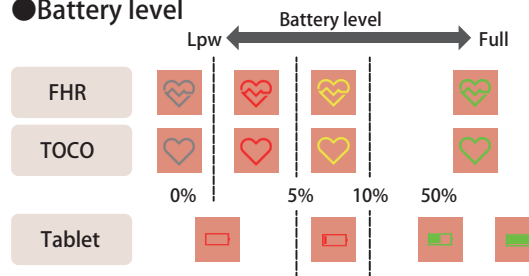

- When the FHC Transducer and UC Transducer can not be recognized by the tablet, it will be displayed the same as 0% battery level.
- A power plug mark is shown while charging.






● Internet connection



Start measurement

1 Power on


- 1 Turn on the tablet
Continue to press until the touch screen shows up
- 2 Wait until the main screen is displayed
- 3 Turn on the FHR Transducer and UC Transducer
Press the power button

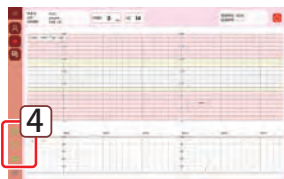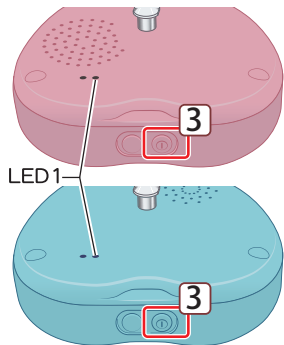
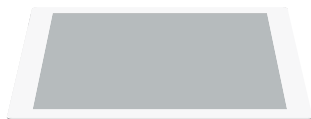


- 4 Check the devices status with the icon on the menu bar
Connect FHR Transducer and UC Transducer to the tablet
Check the batteries of each device
Check the internet connection

See "Indicator (icon) display and function status" (on page 3) for icon display

2 Selecting pregnant woman

- 1 Tap 
Patient list screen will be displayed
- 2 Tap the pregnant woman to measure and select it
- 3 Tap "Select".
Return to the main screen
- 4 The selected pregnant woman will be displayed in the patient page

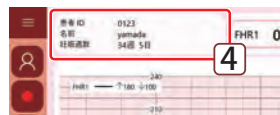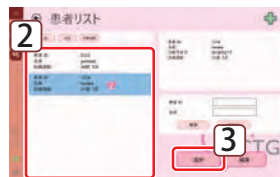


3 Clean up FHR / UC Transducer

Clean up the contact surface of FHR / UC Transducer

4 Wearing FHR Transducer


- 1 Pass the belts (2 pcs) under the waist of the pregnant woman
- 2 Put gel to FHR Transducer

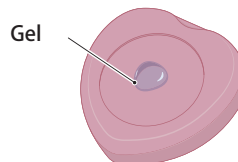



3 Put FHR Transducer on the stomach of a pregnant woman to check the position of the heart of the fetus

4 Place the FHR Transducer at the position confirmed in step 3 and attached belt

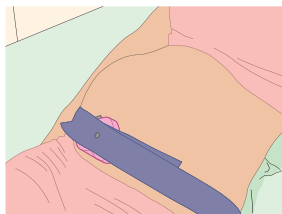


Important

Please fix the FHR Transducer by pulling the belt with equal width on the left and right so that the FHR Transducer will not move to either side of the belt. Also, please do not over tighten the belt.

5 Adjust the volume of the FHR Transducer after consulting with the pregnant woman.

See "Adjusting the volume of the FHR Transducer" (page 2) for the adjustment the volume

5 Wearing UC Transducer

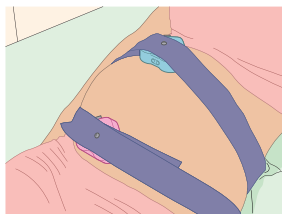


1 Place the UC Transducer on the bottom of the uterus and fasten it with a belt.

Important

Please do not put the gel on the UC Transducer. Please fix the UC Transducer by pulling the belt with equal width on the left and right so that the FHR Transducer will not move to either side of the belt. Also, please do not over tighten the belt

2 Put a blanket to make pregnant woman warm

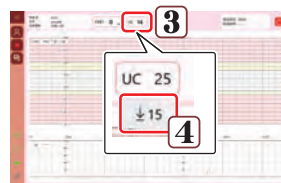


3 Tap the display column of the UC Transducer

Icon for zero reset (process with the current UC Transducer measured value as standard value) is displayed

4 Tap ↓ 15


The UC Transducer measured value is corrected to 15 (standard value)

6 Measurement start

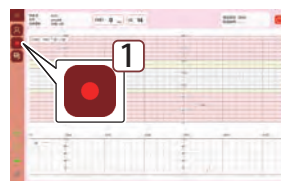

1 Tap

- Measurement time selection dialog box is displayed

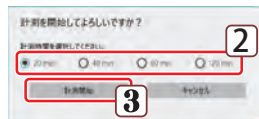

2 Select the measurement time

3 Tap "Start Measurement"


- The background of the screen turns yellow, measurement and data transmission starts.



End measurement


Automatic termination due to measurement time elapsed

When the measurement time elapses, you will hear the end sound to finish the measurement.
After that, a confirmation message will be displayed.

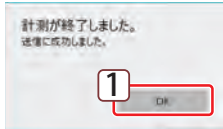


1 Tap "OK"

- Message will be displayed as to whether to shut down

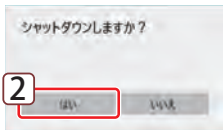


2 Tap "Yes"

- The shutdown process is done and the tablet is powered off
- Tap No if you want to continue use such as browsing past data.
- When you are done, you tap  on the main screen to turn off the power of the tablet.

When data transmission error has occurred

At the end of the measurement, a confirmation message will be displayed

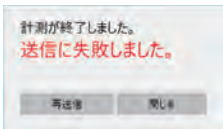


Check the  in the menu bar to see internet status

- Tap "Resend" if it is connected.
- Tap "Close" if it is not connected.
Change the place to connect internet, then resend the data.
See "Resend of data" (page 7) for resend the data.


Stop measurement

To terminate without waiting for the elapsed measurement time, perform the following operation

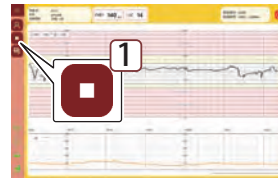

1 Tap

- Confirmation message will be displayed.

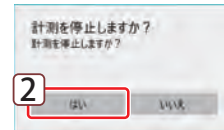

2 Tap "Yes"

- The data is saved and finish transmission, and a confirmation message is displayed. The operation is the same as "automatic termination due to elapsed measurement time"

After completion of measurement

- 1 Remove the FHR / UC Transducer and belts from the pregnant woman
- 2 Wipe off the gel attached to the pregnant woman
- 3 Turn off FHR Transducer and UC Transducer



Press and hold it until the sound is heard (about 3 seconds)
(same as UC Transducer)


- 4 Clean and store the FHR / UC Transducer

Display and transmit past data

Display past data

1 Select patient

- See "Patient Selection" (page 8) for operating manual

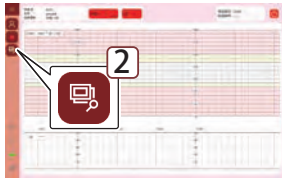

2 Tap


- The past data list of the selected patient will be displayed




3 Tap the data you want to view and select it

- The selected data is displayed.

4 Tap


- The list closes
- To redisplay the list, tap 

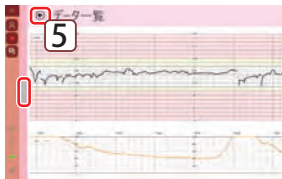

5 After confirmation, tap

- Return to the previous image

Resend data

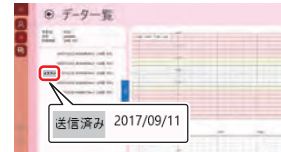

送信済み

is displayed to the data already sent



1 Tap and select the data to send

2 Tap "Resend"

- Press "Yes" to confirm message
- The selected data will be sent
- Message is displayed

Delete data



Deleted data can not be restored



1 Tap and select the data you want to delete

2 Tap "Delete"


- Message is displayed

Patient selection / registration

Display patient list

1 Tap

- Patient list screen will be displayed

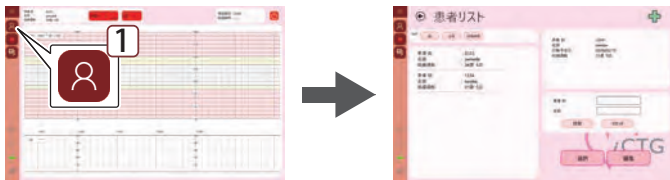

Patient registration


1 Tap

- The patient registered will be displayed in the patient section.

2 Input patient information

3 Tap

- The information is registered and return to the main screen

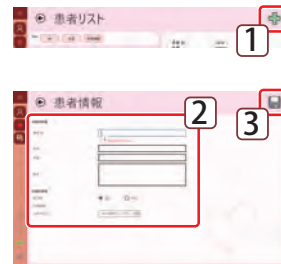

Patient selection



1 If you want to narrow the list, enter patient ID and name and tap "Search"

- Tap "Reset" to cancel the narrowing down

2 Tap the patient to measure and select it

3 Tap "Selection"

- Return to the main screen
The selected patient appears in the patient section.

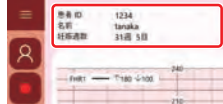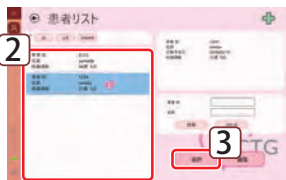

Edit patient information

1 Select patient to modify information



2 Tap "Edit"

- The patient information screen will be displayed

3 Edit patient information

4 Tap

- It registers and returns to the patient list screen. The patient registered will be displayed in the patient section.
- To delete, tap "Delete"
Message is displayed

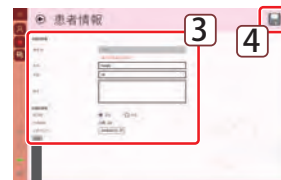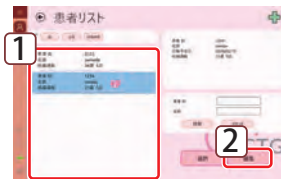

 Deleted data can not be restored